

# ⚠ THIS IS NOT A MEDICAL GRADE MASK ⚠

MADE BY TYNDALE IN THE USA  
WITH FLAME RESISTANT FABRIC


ARC RATING: 9.1 / CAT 2

This mask is made from fabric that is flame resistant and arc rated. It is intended to help slow the spread of SARs-CoV-2 by eliminating the wearer's ability to touch his/her nose and mouth, and by reducing droplets and particulates exhaled or inhaled by the wearer, without posing a flammability hazard. This mask is reusable.

## PROPER USE



Sanitize your hands before donning the mask, to avoid transferring virus onto the mask.

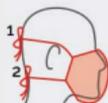

Proper wearing is very important. Fasten the top ties snugly behind your head first, then the bottom ties. The top of the mask should rest on the bridge of your nose, and the bottom should cover your chin, resting underneath the chin. The mask should be fit as snugly as possible.



Breathe normally through the mask.



Proper removal procedure is extremely important, to avoid inhaling anything that may have been deposited on the exterior surface while wearing. Tilt your head forward. Then untie the top straps first, and keeping your head tilted forward, let the mask fall forward, away from your face. Then untie the bottom straps.



Sanitize your hands immediately after removing the mask.

## PLEASE WASH MASK BEFORE FIRST USE

This mask is reusable. It must be cleaned and disinfected before re-use. Wash using standard home laundering with detergent and water, then dry thoroughly. This will both disinfect and clean the mask; chemical disinfectants are not necessary, and their fumes may pose an inhalation risk to the wearer. Do not use bleach or fabric softener. The mask can be laundered with other garments. Flame resistance is not compromised by laundering.



For more on mask safety, donning and doffing, watch our instructional video:  
<https://vimeo.com/tyndalefr/masksafety>



**TYNDALE**  
PROUD TO PROTECT